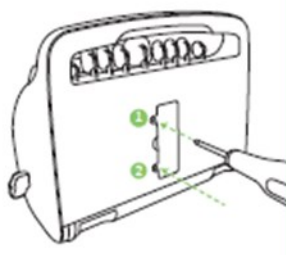
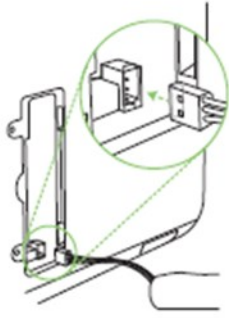
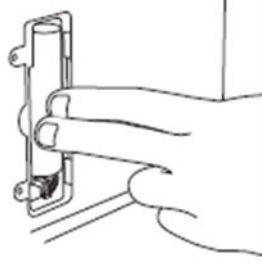
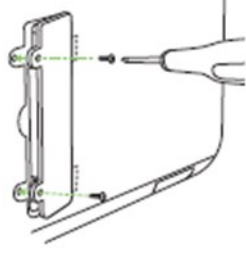
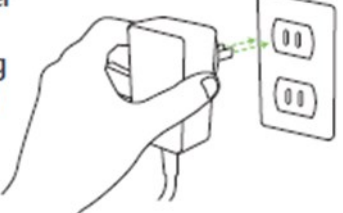
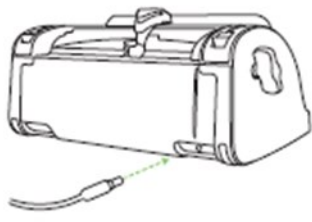
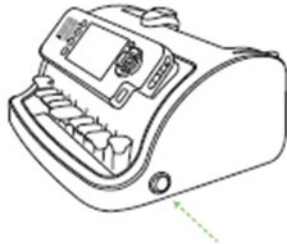



## How to Install the Battery

Warning: Do not plug in the braille until the battery has been installed. To avoid risk of personal injury, do not install the battery if it is damaged.

|   |  |
|---|--|
| <p><b>1</b></p> <p>Locate battery door on bottom, unscrew the two screws and remove door.</p>                | <p><b>2</b></p> <p>Connect battery power plug in the right corner of the battery compartment. The plug fits into the port in only one way.</p>  |
| <p><b>3</b></p> <p>Once connected, place the battery in the compartment and hold in place.</p>              | <p><b>4</b></p> <p>Close the battery cover, first inserting the tabs into the slots on the right. Screw the two screws back in place.</p>      |
| <p><b>5</b></p> <p>Locate power adapter in box and plug adapter into wall power socket.</p>                | <p><b>6</b></p> <p>Plug adaptor cord into its port at the left rear side of braille.</p>   |
| <p><b>7</b></p> <p>Turn on the braille using the switch on the lower right side. Listen for the beep.</p>  | <p><b>8</b></p> <p>Wait 25 seconds for the software to load. When you hear "Welcome" you are in the Welcome Menu.</p>                          |

Fully charging the battery should take no longer than 8 hours. You can use the SMARTBraille™ while it is charging. When charging is complete, the charging icon on the screen will go away.